

Extensive Cleaning Procedure

Materials needed: Clorox bleach and a clean syringe.

1. Prepare 10ml of regular Clorox bleach (do not diluted).
2. Using the up and down arrow keys to move cursor, go to **4.SERVICE** and press enter to confirm.
3. Open cover door with key.
4. Select **No. 2** for **DRAIN CHAMBERS** and press enter to confirm.
5. Pour **5 ml** of **100%** Clorox bleach into the mixing chamber.
6. Pour **5 mls** of **100%** Clorox bleach into the RBC chamber.
7. From the **SERVICE MENU** select **No. 5** for **TRANSFER**.
8. When the transfer is complete, select **No. 1** for **BACKFLUSH** and repeat this step **12 TIMES**.
9. Let the unit sit for a minimum of 1 hour, but longer is better. *** Please note that the longer bleach sit in lines the better chance of clog removal—OVERNIGHT is ideal!** Leave machine on if O/N is performed.*
10. Select **No. 1** for **BACKFLUSH** and repeat **12 TIMES**.
11. Select **No. 4** for **CONCENTRATED CLEANING** and run this once at 100% bleach.
12. Re-run “startup”.